## HANDBOOK + DVD

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**Application techniques of kinesiologic taping in physiotherapy**  
This manual provides the necessary elements to achieve a wide range of applications specific to injuries and the skills required to accomplish the proper application of neuromuscular bandage in the presence of injuries. A practical book, clear, concise and easily to understand, complete with illustrations and a dvd, to realize the neuromuscular bandage in physiotherapy. A key tool for professionals working in functional rehabilitation, prevention and treatment of injuries. Languages: Italian / English.

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**Wrist - 1.** Place the center of the tape at the back side of the wrist extending fingers as much as possible. **2.** Apply it around the wrist leaving a small amount of space between the ends of the tape on the palm.

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**Back Pain - 1.** Place the base of the Y-shaped Physio Kin over the center of the sacrum when standing. **2.** Adhere both ends of the Y-shaped Physio Kin along the muscles of the spine while bending forward.

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**Knee - 1.** Supine with knee fully extended. Apply base of Y-shaped Physio Kin at the upper thigh muscle. **2.** Gradually flex the knee and at maximum flexion, affix the tails around the patella.

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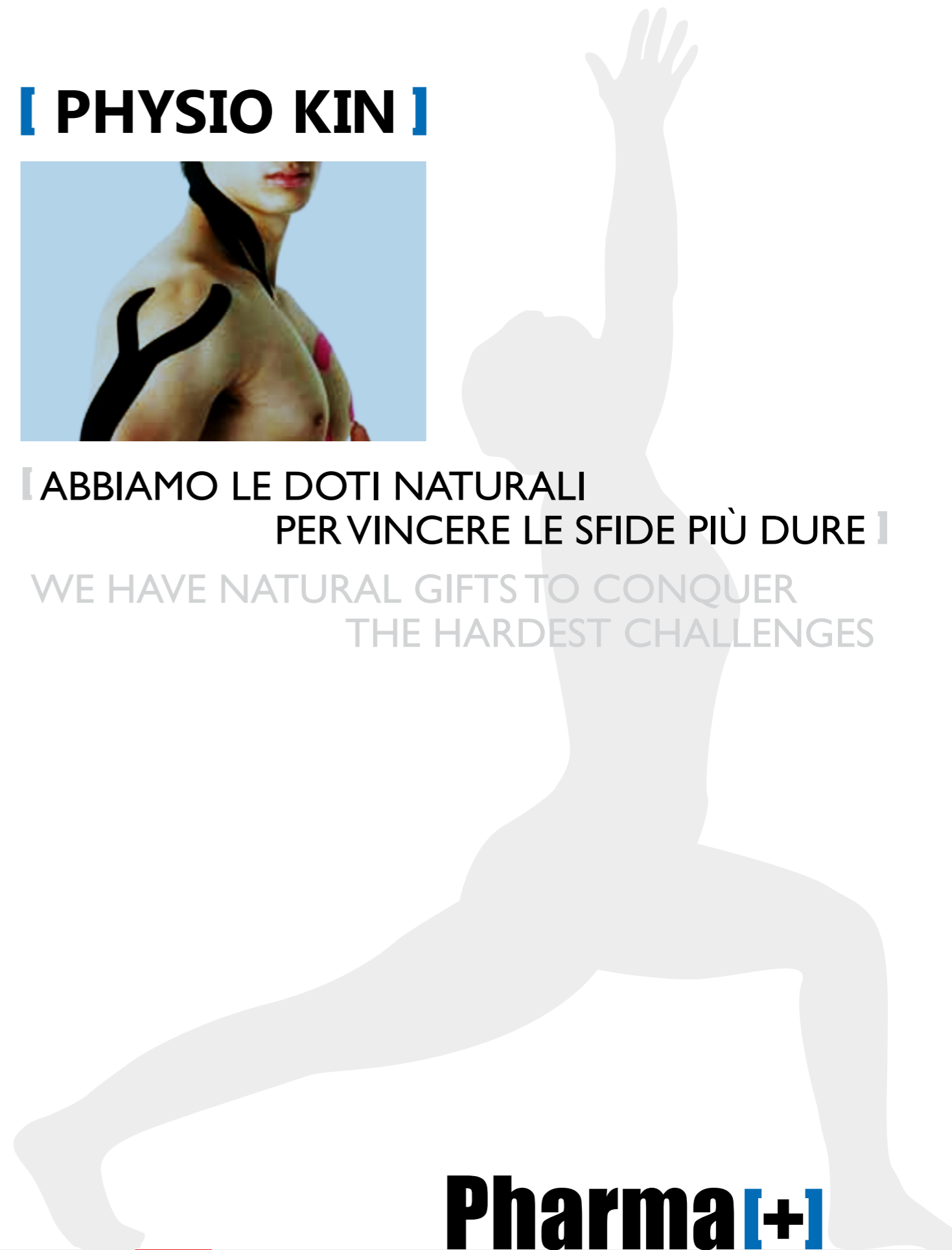
WE HAVE NATURAL GIFTS TO CONQUER  
THE HARDEST CHALLENGES

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http

www.pharmapiu.com

in collaborazione con  
in partnership with

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**Origin\_** The Japanese chiropractic Dr. Murai had ideated, in the 70s, an elastic adhesive tape which was able, through its application, to induce the adaptation of the deep tissues in order to improve movement. Ten years later another Japanese chiropractic, Dr. Kenzo Kase, deepened and developed Dr. Murai's idea giving life to Kinesiology Taping, which soon became popular in many countries, improving notably the approach to many pathologies and medical conditions. It was an innovative method allowing to obtain the best global therapeutic results, based on processes of natural healing of the body through the application of this elastic tape which interacts, thanks to its characteristics, with the elasticity of the skin, taking advantage of the movement in order to obtain the desired therapeutic effect. Hence the origin of the name Kinesiology Taping (in English Tape = Strip; in Greek Kinésis = Movement).

**Diffusion\_** Cardiovascular diseases as well as degenerative-chronic diseases have been increasing since the 80s: this is due to many factors, but firstly by the increase in life expectancy; in the meantime much more people of all age, gender and different social classes have started to practice sports. It is estimated (according to Istat national statistics) that 20% of the general population practices a sports discipline which is usually done without a supervision and therefore a syndrome of overuse can emerge with consequent injuries; furthermore updated statistics relate an increase of work injuries and car accidents. These causes have generated an increasingly high demand of rehabilitation medicine.

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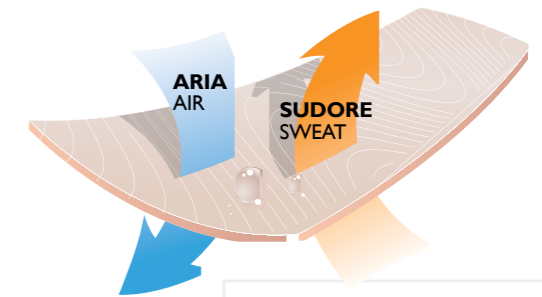
### Indications

PHYSIO KIN may be used in different situations like post-operative supplementary treatments, in rehabilitation, in sports medicine or to improve the lymphatic drainage offering the physiotherapist a new approach to the root of every pathology for the care of the muscles, nerves and organs in the post-traumatic events, in physiotherapy or simply to improve sports performance. PHYSIO KIN can be applied effectively for the treatment of the following pathologies and conditions: vertebral pathologies, disc pathologies, tendon, joint and muscle problems, arthritis, headaches, post-operating pathologies and conditions, neurologic, paediatric and respiratory problems.